

## CP Stent™ Foreshortening Chart

INFLATED BALLOON DIAMETER	CP8Z16 (LENGTH AFTER EXPANSION) (% SHORTENING)	CP8Z22(LENGTH AFTER EXPANSION)(% SHORTENING)	CP8Z28(LENGTH AFTER EXPANSION)(% SHORTENING)	CP8Z34(LENGTH AFTER EXPANSION)(% SHORTENING)	CP8Z39(LENGTH AFTER EXPANSION)(% SHORTENING)	CP8Z45 (LENGTH AFTER EXPANSION)(% SHORTENING)
12mm	1.61cm (2.8%)	2.18cm (0.8%)	2.62cm (4.4%)	3.23cm (3.1%)	3.72cm (1.9%)	4.17cm (3.8%)
14mm	1.54cm (6.5%)	2.08cm (5.4%)	2.56cm (6.8%)	3.15cm (5.4%)	3.66cm (3.6%)	3.97cm (8.4%)
15mm	1.51cm (8.5%)	2.02cm (7.9%)	2.51cm (8.6%)	3.10cm (7.0%)	3.54cm (6.6%)	3.94cm (9.2%)
16mm	1.48cm (10.6%)	1.98cm (10.1%)	2.45cm (10.7%)	3.00cm (9.8%)	3.48cm (8.2%)	3.84cm (11.4%)
18mm	1.43cm (13.7%)	1.89cm (14.0%)	2.38cm (13.3%)	2.88cm (13.5%)	3.20cm (15.6%)	3.71cm (14.5%)
20mm	1.32cm (20.0%)	1.80cm (17.9%)	2.30cm (16.3%)	2.63cm (20.9%)	2.96cm (21.9%)	3.27cm (24.7%)
22mm	1.23cm (25.4%)	1.67cm (23.9%)	2.09cm (24.0%)	2.46cm (26.0%)	2.85cm (25.0%)	3.15cm (27.3%)
24mm	1.05cm (36.4%)	1.46cm (33.8%)	1.91cm (30.3%)	2.07cm (37.9%)	2.27cm (40.1%)	2.83cm (34.9%)